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Information Guide

Respiratory Semi-Mask Type - FFP2


Intended for use by healthcare professionals and patients (level 1)
This product is an Personal protective equipment (PPE).

Image merely illustrative


| | |
|---------------------------------|---|
| Manufacturer | Novavet |
| Adress | Rua Arquitecto Viana de Lima nº 67, 5300-678 |
| City | Bragança, Portugal |
| Trade mark | Bemed |
| Regulation ASAE | - |
| Year of manufacture | 2021 |
| Color | White |
| Dimensions/Size | 156 x 107mm/Single |
| Product Number | 1153247 |
| Model | 5 Ply, Self-Filtering |
| Product Classification | EPI - CLASSE FFP2 BFE ≥94% |
| Material | 5 TNT Layers, Double Coated Wire, Recycled Polyamide |
| Standards and Directives | FFP2 Non Reusable particle filtering half mask without valves according to EU standard EN 149:2001+A1:2009, Regulation N° 14-E/2020 |

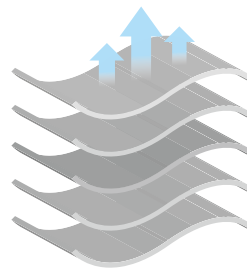Composition

| |
|--------------------------------------|
| Spun-Bond (30g/m ²) |
| Hot-Air-Cotton (50g/m ²) |
| Melt-Blown (25g/m ²) |
| Melt-Blown (25g/m ²) |
| Spund-Bond (30g/m ²) |

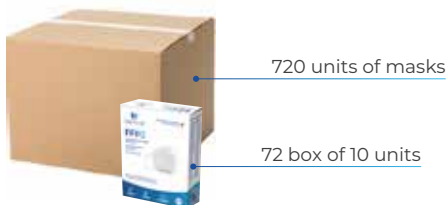Logistics

additional Information

Applicable
Temperatures
vary between 5-50°C
Humidity
relative <80%

Recommended to
change mask
every 8 hours

Expiration date
2 years

instructions For Use

Filter half-masks for protection against particles - FFP2 - against biological risk agents group 2.

The mask protects the user from spills of droplets, infections and contaminating liquids in the environment.

Filters at least 94% of airborne particles.

1. Before putting on the mask, wash your hands with soap and water.
2. Cover mouth and nose with mask and make sure there are no gaps between your face and the mask.
3. Remove the mask by the elastic bands. (don't touch the front of the mask).
4. Replace the mask with the new one as soon as it is damp and do not re-use single-use masks.

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